

TransFix[®] Vacuum Blood Collection Tubes

Product Description

TransFix[®] is a patented whole blood stabilisation solution developed by Sheffield Teaching Hospitals NHS Foundation Trust and manufactured under licence by Cytomark Ltd. TransFix[®] Vacuum Tubes are CE marked.

TransFix[®] Vacuum Blood Collection Tubes are used as a system to collect and stabilise venous blood at the point of collection.. They can also be used to transport and process blood for testing serum or whole blood in a clinical laboratory.

TransFix[®] Vacuum Tubes are plastic tubes of 3ml or 10ml volume with a purple stopper.

TransFix[®] Vacuum Tubes can be used for routine immunohaematology, flow cytometry testing and blood donor screening.

TransFix[®] Vacuum Tubes contain K3EDTA and 0.6ml of TransFix[®] sufficient to stabilise 3ml of blood.

Intended Use and Principles of Stabilisation

The active components of TransFix[®] will stabilise cellular antigens for up to 10 days. In whole blood immunophenotyping and immune monitoring TransFix[®] is added to whole blood at a ratio of 200µl per ml (1:5). The preparation is then treated as a "routine" sample for immunophenotyping procedures and will maintain its staining profile for up to 10 days when stored at 4°C.

Statement of Warnings

1. TransFix[®] Vacuum Tubes treated specimens, samples and all material coming in contact with them should be handled as if capable of transmitting infection and disposed of with proper precautions.
2. TransFix[®] Vacuum Tubes should be store at room temperature (2-8°C). **Do not freeze.**
3. Blood samples mixed with TransFix[®] should be stored at 2-8°C.

4. Avoid contact of TransFix[®] and TransFix[®] treated samples with skin and mucous membranes. If contact occurs immediately wash off with cold water.
5. Do not use TransFix[®] Vacuum Tubes beyond the expiration date printed on the tube.
6. Incubation times or temperatures other than those specified may give erroneous results.
7. TransFix[®] does not contain anti microbial reagents. Microbial contamination must be avoided or erroneous results may occur.
8. Use Good Laboratory Practices (GLP) when handling this reagent.

Storage and Stability

Store TransFix[®] Vacuum Tubes at 2-8°C. Contents of unused tubes are stable until the expiration date printed on the label.

Evidence of Deterioration

The normal appearance TransFix[®] is that of a clear pale green/blue liquid. Precipitation and clouding may indicate deterioration and the reagent should not be used.

Materials Required but Not Supplied

1. Pipettors and plastic tips capable of delivering 200µl to 1000µl.
2. Plastic storage rack.
3. Labels to indicate samples contain TransFix[®] reagent.

Procedure

1. No preparation is needed. Use Vacuum Tubes directly. Insert the Vacuum tube into the tube holder (plastic cylinder)
2. Determine volume of blood in the blood collection tube. A full blood collection tube will contain 3ml of blood.
3. Gently invert 8-10 times after collection to distribute the TransFix[®] reagent throughout the blood sample.
4. Place sample in a rack for subsequent cellular antigen assay. Vacuum Tubes with TransFix[®] are stable for up to 10 days at 4°C, 7 days at Room Temperature (18°C to 24°C) and for 3 days at 37°C. .

Please Note:

1. Samples stored at 4°C must be allowed to return to ambient temperature before any cellular antigen assays are performed.

2. Stored TransFix[®] Vacuum Tubes must be mixed thoroughly by inversion (25x) to re-suspend cells before analysis by flow cytometry.
3. Cell counts from TransFix[®] samples must be divided by 0.8 to adjust for the dilution factor.

Limitations

When using TransFix[®] with samples collected from leukaemic patients it may be necessary to establish an appropriate volume of reagent to add, which may be different to that quoted in the procedure. Volumes in the region of 0.5-1.0ml per ml of blood may be more appropriate.

References:

TVT-103	3ml Vacuum Tubes	10 tubes
TVT-253	3ml Vacuum Tubes	25 tubes
TVT-503	3ml Vacuum Tubes	50 tubes
TVT-1003	3ml Vacuum Tubes	100 tubes
TVT-104	10ml Vacuum Tubes	10 tubes
TVT-254	10ml Vacuum Tubes	25 tubes
TVT-504	10ml Vacuum Tubes	50 tubes
TVT-1004	10ml Vacuum Tubes	100 tubes

Cytomark is a Division of Caltag MedSystems Ltd, a Company registered in the UK, Company number 4162330.

Caltag Medsystems Ltd
Whiteleaf Business Centre
11 Little Balmer
Buckingham
MK18 1TF, UK

Tel: +44 (0) 1280 827460

Fax: +44 (0)1280 827466

Email: office@caltagmedsystems.co.uk

Web: www.cytomark.co.uk